

Study ID _____ Collection Date: MONTH _____ DAY _____ YEAR _____ Collection Time: _____

Sample Timepoint:

- 1 month 3 months 6 months 9 months 12 months
 15 months 18 months 21 months 24 months

Consistency:

- Watery Soft Formed Hard

Color:

- Yellow Orange Green Brown Clay Colored

Amount in diaper:

- Smear Up to 25% 25-50% More than 50%

Instructions:

After you have finished changing your baby's diaper, please collect the sample immediately following your baby's bowel movement.

1. Holding the yellow tube top, unscrew ONLY the purple cap from the kit and set aside
2. Use the spatula to collect a pea-sized amount of stool into the top yellow portion of the tube. (Do NOT push sample into the plus-sign opening.)
3. Tightly screw back on the purple top
4. Shake the tube in a back and forth motion until stool dissolves
5. Place sample in the clear bag. **Do NOT freeze this sample.**
6. Fill out the back of this form
7. Immediately email us at **TheMagicStudy@email.chop.edu** to schedule courier pick up



Questions? Call 267-223-9730 or email TheMagicStudy@email.chop.edu